

CE
2163



FDA
CLEARED

Adult
use 1pcs

WHITE

Air
QUEEN

Breeze Mask

EN 149:2001 + A1:2009
FFP2 NR

ISO 13485



Nanofiber Filter Mask



Thin and light

Light fiber with excellent air-permeability
-breathable and prevents contaminants



Nanofiber filter

Intense filtering effect and Fiber with
excellent air-permeability applied



Ergonomic design

Prevents slipping down and steaming
up with nose support design

Air QUEEN Breeze Mask

How to use the product



1 Wear the mask to cover your nose and chin.



2 Hook each strap over your ear and press the nose piece with the fingers to lock them in place



3 Hook each strap over your head with a ring.



4 Make sure the mask is in close contact with your face without air leakage.

Precautions for Use

- ① Make sure not to place towel, tissue, etc. inside of the mask.
- ② Do not use if the inside of the mask is contaminated.
- ③ Do not wash for use.
- ④ Do not crush or deform the mask.
- ⑤ Do not touch the surface of the mask after wearing.
- ⑥ If you are a pregnant women, a child, a senior citizen, a patient suffering from respiratory tract or cardiovascular disease and feel uncomfortable in breathing when you wear the mask please discontinue using immediately and consult your doctor or other specialist if necessary.

Product Information


Product Name : Air QUEEN Breeze Mask	
Type of Mask: Filtering half masks to protect against particles	
Quantity : 1 piece	Storage: -20 °C ~ 30 °C
Ingredients : nonwoven(outer, lining), filter(nanofiber), plastic coated wire, polypropylene ring, nylon strap	
Customer Service : +82-41-629-9500	Expiration Date : 3 Years From The Manufacturing Year/ Month

Manufacturer : TOPTEC Co., Ltd. 122, Asanvalley-ro, Dunpo-myeon, Asan-si, Chungcheongnam-do, Republic of Korea

Supplier : TOPTEC Co., Ltd. 122, Asanvalley ro, Dunpo myeon, Asan-si, Chungcheongnam-do, Republic of Korea

Supplier : LEMON Co., Ltd. 1105-65, Sanho-daero, Sandong-myeon, Gumi-si, Gyeongsangbuk-do, Republic of Korea

If there is something wrong with this product, it can be exchanged and refunded according to the consumer dispute resolution standard announced by the Fair Trade Commission.

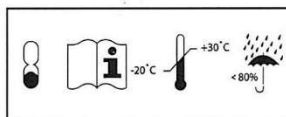

EXP: 2025/02 PRD: 2022/03
LOT: T220209-FFP2WL

Made in KOREA

