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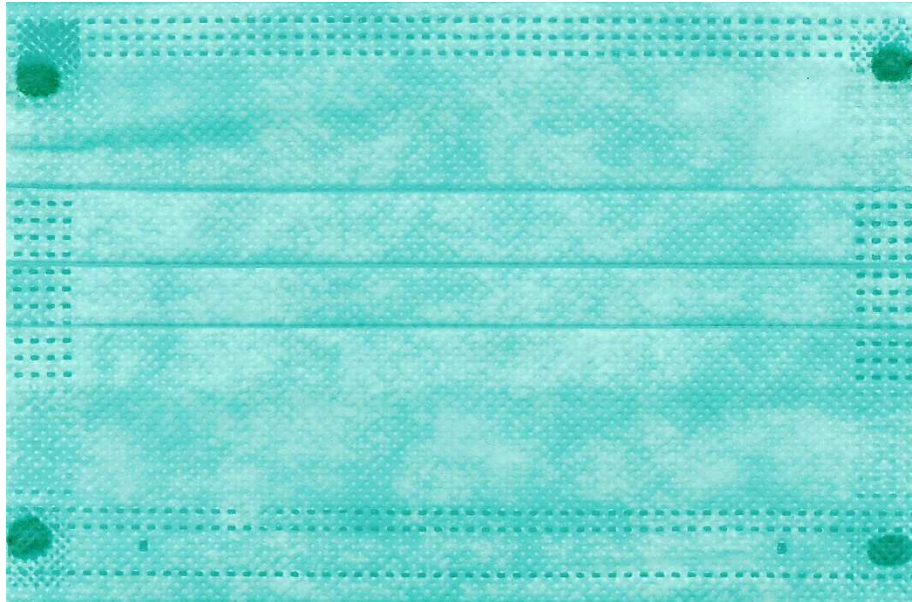
**EN**  
The mask is intended to  
minimise the release of  
aerosols and protect  
against aerosols. For  
single use only. Wash  
hands thoroughly before  
putting on the mask.

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Make sure that the  
coloured side is worn on  
the outside and the nose  
clip is fitted at the top. Put  
on the mask and press the  
nose clip down so that it  
adjusts to the contours of  
your nose. Fix the ear-  
straps securely behind the  
ears.

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Place the mask in such  
a way that it completely  
covers your entire  
mouth and nose. Only  
then will the mask fit  
properly. While wearing  
the mask, try not to touch  
it. Replace and dispose  
of the mask when it  
becomes too damp.

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Remove the mask with  
clean hands at the ear  
bands. Dispose the mask in  
a closed container and in  
accordance with legal  
requirements.  
**ATTENTION!** Do not use  
under the following  
conditions: Children under  
the age of 16 years. In case  
of prolonged breathing  
difficulties,  
intolerance/allergies to  
named fabrics, Mechanical  
injury to the skin. Medical  
examination.

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Surgical Mask (Medical)

14.5 × 9.5 cm Typ IIR

EC Medical Devices Registration Number: RPS/998/2020

EC REP

WeMoove e.U.  
Salzbergstr. 82, 6067 Absam  
Österreich [www.physioshop24.net](http://www.physioshop24.net)

MD

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Hunan Huaxiang Industry Co., Ltd